

PROFESSIONAL
HEARING

S E R V I C E S

Division of Moreland Ear, Nose and Throat Group, Ltd.

December 2009

Holiday Edition



*Wishing You
a Wonderful Holiday Season
And a Happy New Year*



*Tracey, Kiz, Lisa,
Jen and Heather*



Holiday Gift Baskets and Bags Now Available

A variety of pre-made holiday baskets and bags are available at our front desk.
Baskets include a variety of our most requested supplies
such as: batteries, hearing aid savers, sanitizers, earplugs, wax guards and more
Prices range from \$13 to \$35



PROFESSIONAL
HEARING
 SERVICES
 Division of Moreland Ear, Nose and Throat Group, Ltd.

1111 Delafield St. Suite 102, Waukesha
 13900 W. National Ave, New Berlin
 240 Maple Ave., Mukwonago
 1185 Corporate Dr., Oconomowoc

Phone: 262-549-5150
 Fax: 262-549-1337



Living Longer: How to get the most out of your hearing aids

Here are 4 simple steps to care for your hearing aids:

1. Clean your hearing aids daily: Unlike your body, hearing aids should never be cleaned with water. Use the cleaning tools provided with your hearing aids. Follow the directions in your user manual to ensure proper cleaning. Remember, it is important to keep debris away from the openings of the hearing aids.
2. Keep your hearing aids dry: Moisture is one of the leading causes of damage to hearing aids, especially in the humid summer months. A hearing aid dehumidifier, such as a Hearing Aid Saver, should be used daily when not wearing the hearing aid. The Hearing Aid Saver will dry out the hearing aids digital circuitry while you sleep. The dehumidifiers are inexpensive and an easy way to extend the life of your hearing aids.
3. Periodic Maintenance: Just like changing the oil in your car, hearing aids do require both monthly and yearly maintenance. Hearing aids with domes and wax guards should have these parts replaced monthly. Behind-the-ear devices with standard earmolds should have the tubing replaced annually. Failure to perform regular maintenance, will result in decreased performance such as muffled sound, static, intermittency, and whistling.
4. Pet & Loss Prevention: Animals love the smell of their owner and are drawn to hearing aids. The easiest way to prevent hearing aid theft and damage from your pet is to keep them in their case when not in use. Using the case when not in use will also help with misplacement of hearing aids.

COMPLETE THE WORD SEARCH & RETURN TO OUR OFFICE
 TO BE ENTERED IN A DRAWING FOR A FREE DELUXE GIFT BASKET
 DRAWING WILL BE HELD ON DECEMBER 31, 2009

U	T	L	S	O	S	E	T	A	A	U
T	S	I	G	O	L	O	I	D	U	A
D	E	C	N	A	L	A	B	S	D	E
E	C	A	I	N	A	A	I	L	I	L
C	G	T	R	C	I	G	E	A	O	H
I	B	C	A	N	E	T	P	S	G	C
B	A	H	E	L	A	R	U	L	R	O
E	S	L	H	E	U	T	U	S	A	C
L	T	T	E	E	A	T	C	M	M	U
O	L	R	A	O	S	T	A	P	E	S

EAR
 COCHLEA
 BALANCE
 STAPES
 HEARING
 DECIBEL
 AUDIOLOGIST
 AUDIOGRAM
 TINNITUS
 CERUMEN

NAME: _____

ADDRESS: _____

E-MAIL: _____

PHONE: _____